




## **Tritanium® PL** Posterior Lumbar Cage

### Developed to **minimize subsidence**

Subsidence is a known risk with any lumbar interbody fusion device. In addition to bone quality, endplate preparation, and cage position within the disc space, features of the device itself can help to minimize subsidence.

Features of the Tritanium PL Cage that are designed to minimize subsidence:<sup>1</sup>


- **Precisely engineered material modulus**
- **Maximized surface area**
- **Optimized cage geometry**

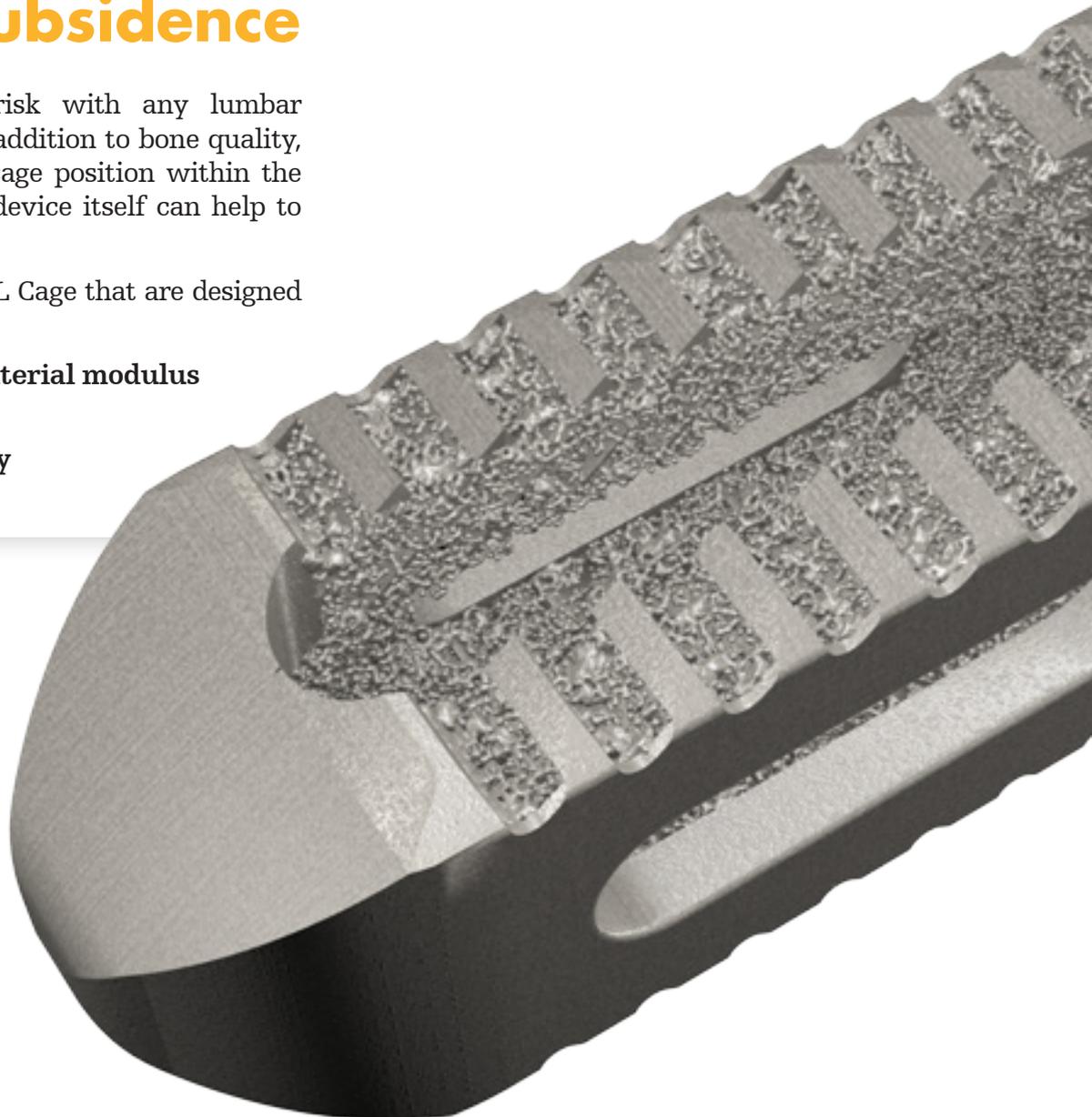

## Precisely engineered material modulus

The porous nature of Tritanium gives it an elastic modulus that falls between cancellous and cortical bone, the two types of bone that form vertebral bodies.


**Tritanium demonstrated an elastic modulus lower than that of other materials.<sup>1</sup>**

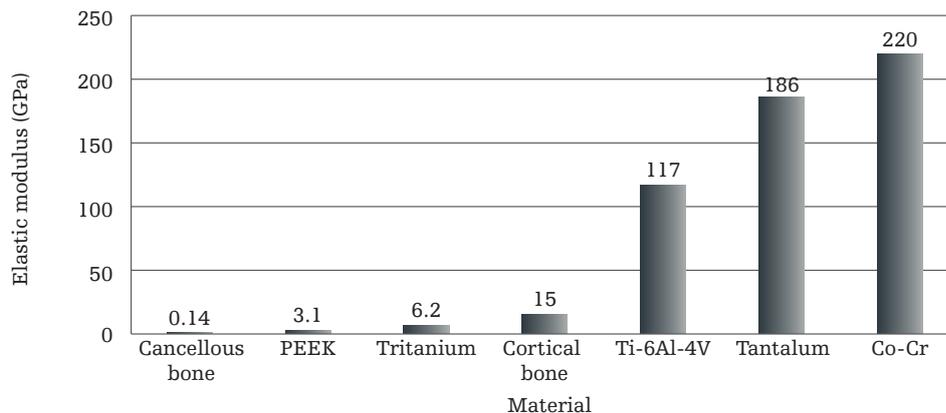

## Maximized surface area

The superior and inferior teeth of the Tritanium PL Cage have been designed to increase the total surface area of the device in contact with bone, to normalize the load transmission and minimize subsidence.


**Teeth of the Tritanium PL Cage were shown to have a greater surface area than teeth of the same height from a PEEK cage.<sup>1</sup>**

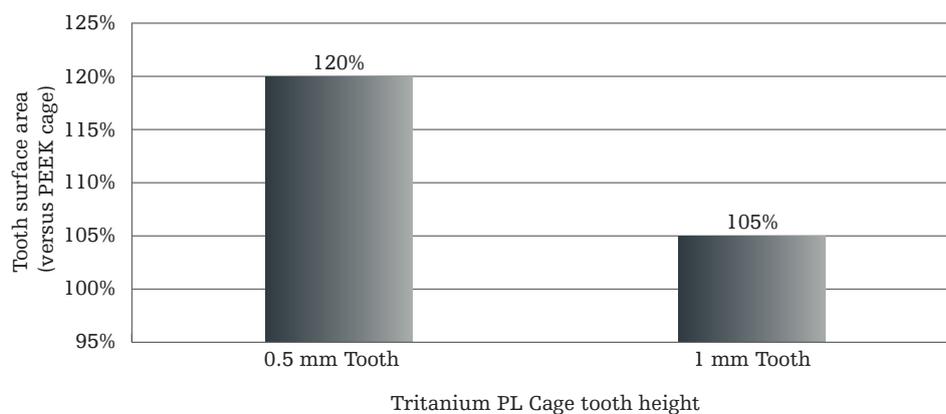

## Optimized cage geometry

The Tritanium PL Cage was designed with a large central graft window and two lateral windows to reduce the overall stiffness of the cage<sup>1</sup> and minimize subsidence.

**The Tritanium PL Cage demonstrated better resistance to subsidence than other commercially available posterior lumbar interbody cages constructed out of different materials, including those with a larger footprint.<sup>1</sup>**

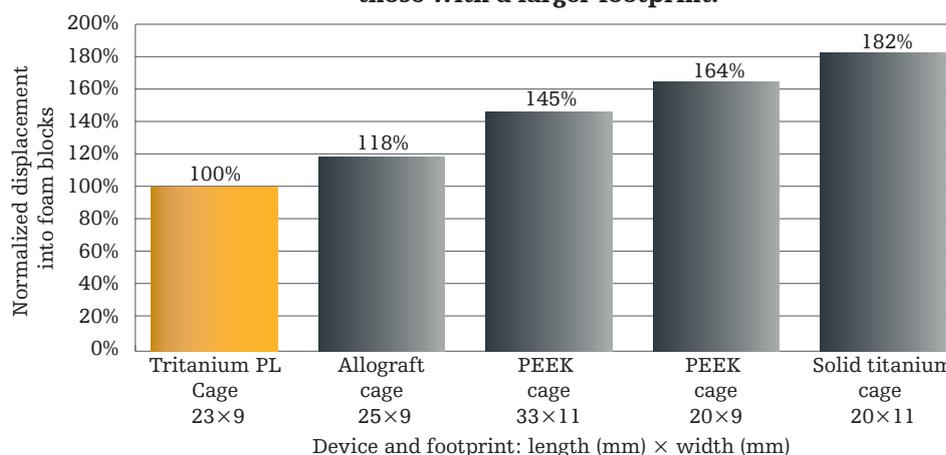

Subsidence was measured at 500 N of compressive force. Testing was performed per ASTM F2267.

### References

1. Subsidence summary PROJ\*42624.

A surgeon must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that surgeons be trained in the use of any particular product before using it in surgery.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A surgeon must always refer to the package insert, product label and/or instructions for use before using any Stryker product. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker, Tritanium. All other trademarks are trademarks of their respective owners or holders.

TRITA-SS-1

Copyright © 2016 Stryker



**Stryker's Spine Division**  
 2 Pearl Court  
 Allendale  
 NJ 07401-1677 USA  
 Phone: 201-749-8000  
[www.stryker.com](http://www.stryker.com)